

**Product And Manufacturer Information**

- Contents:** 1 – Reference Electrode
- Intended Use:** Use with Diamond Prolyte, Medica EasyLyte, IL ILyte and Menarini SpotLyte Systems.
- Storage:** Store the Reference Electrode in the protective packaging and vinyl cap provided with the electrode at room temperature (18-25°C) until you are ready to use it in your system. Make sure that the transport housing is filled with reference electrolyte solution (you may use the reference electrolyte solution remaining in the reference electrode housing at the time of disassembly).
- Precautions:** Never rinse the reference electrode with distilled water. Never allow the reference electrode to become dry.

Diamond Diagnostics, 333 Fiske Street, Holliston, MA.  
**For Technical Assistance call:**  
Diamond Diagnostics Technical Services at 1-508-429-0450

**Installation**

- 1 Remove electrode from the protective packaging.
- 2 Remove vinyl cap.
- 3 Using the Reference Electrode tool supplied in the Troubleshooting Kit, screw the new Reference Electrode into the Electrode Housing.
- 4 Do not over tighten the Reference Electrode.
- 5 Perform a two-point calibration.
- 6 Analyze a quality control sample.

**Warranty**

Warranty is calculated 12 months from the date of shipment from Holliston, MA.

